

SAFE MASK



NOT YOUR AVERAGE TEXTILE SOCIAL FACE MASK!!



COMFORT



ECO-FRIENDLY



ANTI-FOG



MODERN DESIGN

How to position the mask correctly



How to safely remove the mask



STEIN

www.steinworldwide.com

PRODUCT CHARACTERISTICS

- **COMMUNITY SAFETY PRINCIPLE OF PRECAUTIONARY PUBLIC HEALTH ADVISED BY THE PORTUGUESE HEALTH DIRECTORATE**
(information nº 009/2020 of 13/04/2020)
- **ACCORDING TO THE CATEGORY 2 - EN 14683:2019 NORM**
Particles retention (equal or higher than 3µm) ≥ 90%, for up to 4 hours of use, according to the Portuguese Health Directorate technical orientation.
- **THREE LAYERS OF PROTECTION**
Produced in polyester on the outside, and non-woven fabric and polyester on the inside.
- **WASHABLE** (according to NP EN ISO 6330:2012)
Allows machine washing with detergent up to 60°, until 13 washes (after, the mask safety isn't assured). The mask has to be dry to be used again.
- **RE-USABLE**
Less environmental impact compared to disposable options.
- **EASY TO APPLY**
The elastics make the usability easier and more convenient.
- **FRIENDLY LOOK**
Designed to offer a young and friendly look, with many different colour options.
- **ANTI-FOG MEMBRANE**
The soft-nose layer with a nose clip on the top of the mask, prevents the passage of air to avoid fogging the glasses.
- **MAXIMUM COMFORT AND MODERN DESIGN**
- **LOWER ECOLOGICAL FOOTPRINT**

USAGE INSTRUCTIONS


(ACCORDING TO THE PORTUGUESE HEALTH DIRECTORATE Nº019/2020 DE 03/04/2020)

- The mask must be washed before its first utilization.
- Before putting it on, you must sanitize your hands with soap and water or with an alcohol based solution.
- The mask must be placed with the internal side towards your face.
- Adjust the nose clip, making sure that it is well adjusted and that there is no openings.
- If you touch the mask while wearing it, you must sanitize your hands immediately.
- The maximum wearing period recommended is 4 hours a day, or as soon as it is damp.
- If the mask gets wet, you must change it immediately.
- The mask must be washed right after being used.
- The removal must be done through the elastics, to avoid touching the mask.
- The mask must be placed individually in a closed plastic bag, until placed in the washing machine.
- Sanitize your hands in the end of the mask usage and whenever you touch it.
- The mask must be kept in a dry place and protected from possible contamination.
- In any case of defect or damage, do not use it.
- When the mask life cycle ends, wash it and dispose it as non-hazardous textile waste.
- If you decide to not wash it, you must dispose it as biological waste.

WARNING

- The protection is only guaranteed if the usage instructions are fulfilled.
- These masks cannot be used by health professionals on duty.
- If you're sick, these masks are not suitable. Please contact your doctor.
- These masks do not replace other prevention methods, such as social detachment and frequent hands hygiene.
- The masks protection is compromised if they are wet or damped.

MASK SIZING

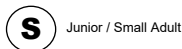

For maximum protection the correct size mask must be worn

TEST REPORT BA - 2832A/2020

TEST	STANDARD	UNITY	RESULT	TECHNICAL SPECIFICATION **
Bacterial filtration efficiency, BFE*	MII 170 (EN14683:2019)	%	79,2	Level 3 ≥ 70%
Differential pressure*	MII 170 (EN14683:2019)	Pa/cm ²	28,0	Level 2 ≥ 90%
				< 40



This mask has a 90% filtration until 13 washes, and goes down to 79% with 25 washes

WASHING INSTRUCTIONS

