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SOLUBLE HEMOSTATIC GAUZE

Sterile contents

Description: WoundClot™ is a soluble gauze, based on cellulose structure. It is fully biocompatible and provided sterile. In contact with liquids it forms an adhesive gel which expands and adhere to the wound for bleeding control.

WOUNDCLOT™
Hemostatic Gauze

Advanced Bleeding Control™

INSTRUCTIONS
FOR USE (CE)

FOR TEMPORARY CONTROL OF BLEEDING

Sterile contents

Product size: 2.5x2.5cm (1x1"), 5x5cm (2x2"), 5x10cm (2x4"), 10x10cm (4x4"), Twin pack- 2.5x2.5cm (1x1") & 5x5cm (2x2").
Indications: of superficial wounds, minor cuts, and abrasions. **Application:** Place the gauze onto the wound, apply slight pressure and let the blood flow through the gauze until it forms into gel and blood stops flowing. If needed, you may slightly wet the gauze to create a complete gel membrane. **Removal:** Once hemostasis is achieved, remove by saturating the gauze with water or saline solution, then gently wipe any excess gauze material away from the wound. **Cautions:** It is advised to clean itching, burning, rash or signs of infection such as fever, pain, redness or swelling, seek professional medical care. WoundClot™ gauze is intended for external bleeding control the wound with water before using WoundClot™. In case of itching, burning, rash or signs of infection such as fever, pain, redness or swelling, seek professional medical care.

- Do not re-sterilize
- WoundClot™ gauze is a single use device. Do not reuse.
- Consult instruction for Use
- Sterilization using Irradiation
- Do not use if package is damaged. Sterility of contents guaranteed unless individual package is damaged or opened.
- Do not use after expiration date.
- Store in dry place
- LOT Batch code
- REF Catalog number
- Manufacturing Date

Manufacturer: **Core Scientific Creations Ltd.**
29 Nahshon Street, 4926918, Petah Tikva, Israel
T: +972-3-7359555 | F: +972-3-7359544
questions@woundclot.com | www.woundclot.com

EC REP EC Rep Authorized Representative in the European Community
EU Authorized Representative: CE partner4U BV, Esdoornlaan 13, 3951 DbMaarn, The Netherlands | Fax: +31 343 442 162
CE Marketing and identification number of notified body.

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Produsor: Core Scientific Creations Ltd.
Manufacturer: Core Scientific Creations Ltd.
Fabricante: Core Scientific Creations Ltd.